



Continuous Relief for Acute and Chronic Dry Mouth

IMPORTANT SAFETY INFORMATION

WARNING:

The Voutia™ System is designed to be used by individuals suffering from a perceived or actual lack of sufficient saliva production. Use by individuals with an impaired swallowing ability, risk of aspiration, impaired mental status, or have failed a physician prescribed swallow test is not recommended. This device is not waterproof. Do not immerse the device or allow it to be in direct contact with fluids to avoid potential damage to the unit. **DISCONTINUE USE IMMEDIATELY IF YOU EXPERIENCE DISCOMFORT OR TROUBLE WITH SWALLOWING, CHOKING OR ASPIRATING.**

When using the Voutia™ System, basic safety precautions should always be observed, including the following:

1. Read all instructions before using the Voutia™ System.
2. Only use the Voutia™ System for its intended use.
3. Do not allow the Voutia™ System to be used as a toy.
4. To protect against the risk of electric shock, do not immerse any part of the Voutia™ System in water or other liquids.
5. Never yank cord to disconnect from any outlet. Grasp plug and pull to disconnect.
6. Do not allow any of the Voutia System components to be in contact with hot surfaces.
7. Always disconnect the Voutia™ System from its power source before filling the bottle.
8. Do not operate the system with a damaged cord or plug, or if the appliance or any other part of the Voutia™ System has been dropped or damaged. To avoid risk of electric shock, do not disassemble or attempt to repair any component of the Voutia™ System. Return the Voutia™ System to the manufacturer for inspection and repair or replacement.
9. Do not leave the Voutia™ System unattended while connected to power sources.
10. Use caution when emptying the unit. Do not allow water to drain into or on the components.
11. Do not operate in the presence of explosive and/or flammable fumes.
12. This device is intended for individual personal use. Do not share the Voutia™ System among different individuals. Follow all instructions included with the unit/system.
13. NEVER use any chemicals or substances in the Voutia™ System other than the recommended cleaning solution or potable water.
14. Use of unauthorized substances can damage the device and void any warranty.

To reduce the risk of fire, electric shock, or injury:

- Do not leave the Voutia™ System plugged in while unattended.
- ALWAYS unplug from power sources when refilling the bottle.
- Use only as described in the enclosed manual or under the written advisement of a licensed Physician or Dentist.
- DO NOT pull cord or tube, use cord or tube as a handle, close a door on the cord or tube, or pull cord or tube around sharp edges or corners. Keep cord and tube away from heated surfaces.
- DO NOT use extension cords or outlets with inadequate current carrying capacity.
- Do not unplug by pulling the cord. To unplug, grasp the plug, not the cord.
- Do not handle the Voutia™ System or plug with hands that are damp or wet.
- Do not put objects into openings or use if any opening is blocked.
- Store the Voutia™ System indoors in a cool dry area. It should not be left in a hot or cold car for any length of time.
- DO NOT use cleaning solutions (other than what is recommended by the manufacturer).
- Do not attempt to fill the Voutia™ System under a faucet. Always use a measuring cup.
- KEEP THE Voutia™ System **OUT OF THE REACH OF CHILDREN. USE FOR AND WITH CHILDREN UNDER 18 SHOULD ONLY BE DONE UNDER THE RECOMMENDATION OF A PHYSICIAN OR DENTIST AND WITH CLOSE ADULT SUPERVISION. FOLLOW ALL ENCLOSED INSTRUCTIONS.**

IMPORTANT SAFETY INFORMATION *CONTINUED*

THE VOUTIA™ SYSTEM IS FOR SINGLE PATIENT USE ONLY

DO NOT SHARE THE DEVICE WITH OTHERS

The device should not be immersed in liquids or exposed to precipitation. Use of the Voutia™ System should not be done in the presence of flammable anesthetic mixture with air or with oxygen or nitrous oxide.

Maximum Time of Continuous Use of The Voutia™ System: 24 hours a day.

This system is designed to deliver potable fluid to the oral cavity. It may be used for any extended period of time. Users should take care to note if the fluid being delivered states a maximum allowable amount within a certain period of time.

DO NOT EXCEED RECOMMENDED FLUID AMOUNTS OR DOSEAGES FROM ANY MANUFACTURER OR SUPPLIER.

ADDITIONAL IMPORTANT SAFEGUARDS.

1. To avoid circuit overload, DO NOT operate another appliance on the same socket (circuit)
2. DO NOT leave the Voutia™ System unattended when the system is connected to a power supply/electrical outlet.

The manufacturer accepts no responsibility for damage caused to objects, materials, animals, or persons due to the inappropriate use of the Voutia™ System, and/or through ignoring the advice and recommendations set down in the Owner's Manual. Specific written recommendations for use by a licensed Physician and/or Dentists should still adhere to the manufacturer's basic operating instructions.

Use of prescription medications or saliva substitutes is considered "off label" use and is not recommended.

The manufacturer accepts no responsibility for damage caused to objects, materials, animals, or persons due to the use of saliva substitutes and/or prescription medications independently or under the guidance and supervision of a licensed health care professional that deviate from use guidelines set forth in this document.

THANK YOU FOR CHOOSING VOUTIA™

Your Voutia™ System is a very versatile device. Used correctly, it is intended to make life more comfortable for individuals suffering from the effects of dry mouth (xerostomia).

**For Customer Support please call
804.803.1840 or email support@voutia.com**



Continuous Relief for Acute and Chronic Dry Mouth

Virginia Head and Neck Therapeutics, Inc.
10149 Bon Air Crest Dr. Richmond, VA, 23235
804.803.1840 | www.voutia.com